



- Grip the HOLD tab with thumb and finger of one hand to stabilize device and securement feet beneath the skin
- Pry upward at the edge of LIFT tab with index finger tip of other hand and thumb placed at center to release cover from base



- Completely detach cover from base



- Remove the catheter - do not use excessive force
- Hold pressure at site to achieve hemostasis
- If the insertion site is "crusty" or firm, apply sterile gauze and saline at the site for a few minutes. The tissue will soften and removal will be easier

FOLD OPTION

Can be used with or without catheter in place



- Apply firm pressure at the insertion site to keep tissue still
- Fold wings downward to bring feet together under the skin



- Hold folded base horizontal to skin
- Use swift, deliberate upward motion to remove following the shape of the feet

SPLIT OPTION

Catheter must be removed prior to using this method



- Use a blunt tip scissors to cut base completely in half lengthwise along the groove



- Apply firm pressure at the insertion site with one hand
- The flexible securement feet are shaped like an "L" with the feet extending 5mm to each side of the insertion site



- Turn blue edge upward and use a swift, deliberate upward motion to remove each foot separately following the shape of the foot

24-Hour Clinical Information Line 1-800-225-0000

Go to www.securacath.com for more information

Please refer to instructions for use for indications, contraindications, hazards, warnings, cautions and directions for use.



181 Cheshire Lane, Suite 100
Plymouth, MN 55441 USA
+1.763.225.6699
www.securacath.com

Download the new SecurAcath® app!



CE 0473

www.securacath.com/patents
Interrad Medical and SecurAcath are trademarks of Interrad Medical, Inc.

©2018 by Interrad Medical, Inc. All rights reserved.
1329-013 Rev. H