

---

# Mobile Review Workflow


---

## **Summary**

This document contains the interpretation workflow using Mobile Review for Physicians. Mobile Review is designed for remote review and interpretation of patient data via the intranet. It allows the user to filter patient visits by date, location, assigned physician, and status of visit.

## Mobile Review Interpretation Workflow

1. Launch a web browser and enter the address: <http://servername:5300/reviewservice/>
2. Enter Login Name and Password.

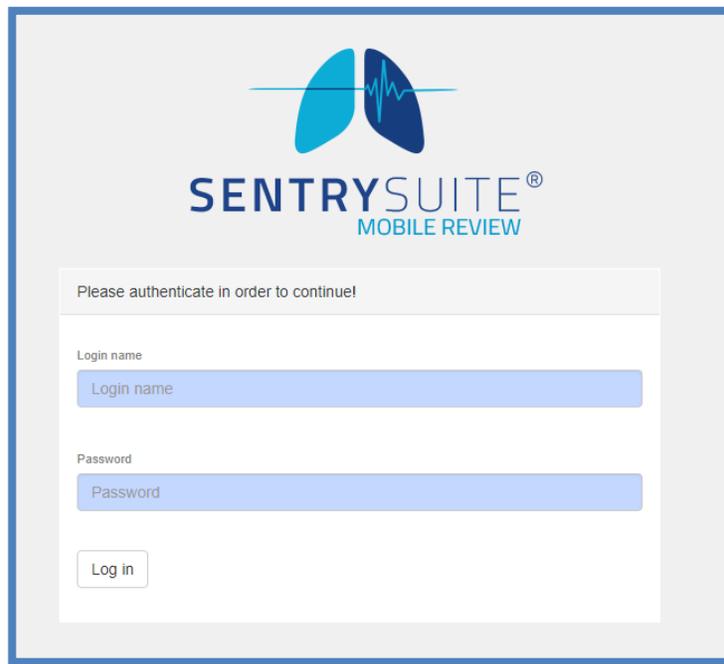

The screenshot shows the login interface for SentrySuite Mobile Review. At the top center is the logo, which consists of two stylized lungs in light blue and dark blue, with a white ECG line passing through them. Below the logo, the text "SENTRYSUITE®" is displayed in a large, bold, dark blue font, with "MOBILE REVIEW" in a smaller, lighter blue font underneath. Below this, a white box contains the text "Please authenticate in order to continue!". Underneath this box are two input fields: "Login name" and "Password", each with a light blue border and a white background. Below the password field is a "Log in" button with a white background and a light blue border.

3. Click the [+ ] icon to expand the filtering options.



The screenshot shows the dashboard of the SentrySuite Mobile Review application. At the top left is the logo. To the right of the logo is the text "SENTRYSUITE®" and "MOBILE REVIEW". In the top right corner, there is a "Doctor" profile icon and a "Log out (00:09:53)" button. Below the logo, there are several input fields: "Date" with a dropdown menu, "Physician" with a dropdown menu, "Location" with a text input field, and "Patient ID" with a text input field. To the right of these fields is a "0 filters active" indicator and a red-bordered box containing a "+" icon. Below the input fields, there are navigation arrows and the text "0 visit(s) found". In the bottom right corner, there are buttons for "Batch Mode", "Save Filter", and "Filter Presets".


- From the Visit Status field, select **[Testing Complete]** and **[In Review]**. If further filter options are needed, filter by assigned **[Physician]** name or **[Location]** by using the drop-down menus. Once all filtering options are selected click **[Use and Hide]**.

**NOTE:** The filter settings are stored upon **Log out**. Therefore, it is not always necessary to reset filters upon **Log in**. However, it is good practice to confirm the filter settings are correct.


- A list of patient visits will be displayed based on your filter settings. “Testing Complete” visit status is represented with a gray colored flag. “In Review” status is represented with a blue colored flag. Below each patient visit will be a report ready for interpretation. Click the **[PDF icon]** or **[report name]** to open a single patient report. Click **[Batch Mode]** if multiple studies will be read at one sitting.

Time	Name	Patient ID	Date of Birth	Status
03/28/2019 6:29 PM	Sample SeS	CFNSPJ0750209	02/09/1975	Testing Complete
	<b>PFTREPORT</b>	03/28/2019	9:17 PM	Preliminary
01/17/2019 11:25 AM	VYAIRE ONE	P1	08/28/1975	Testing Complete
	<b>ADULT_COMPLETE</b>	01/17/2019	11:28 AM	HIS Preliminary


- The report will be displayed on the left side of the screen. The interpretation window will be displayed on the right side of the screen.

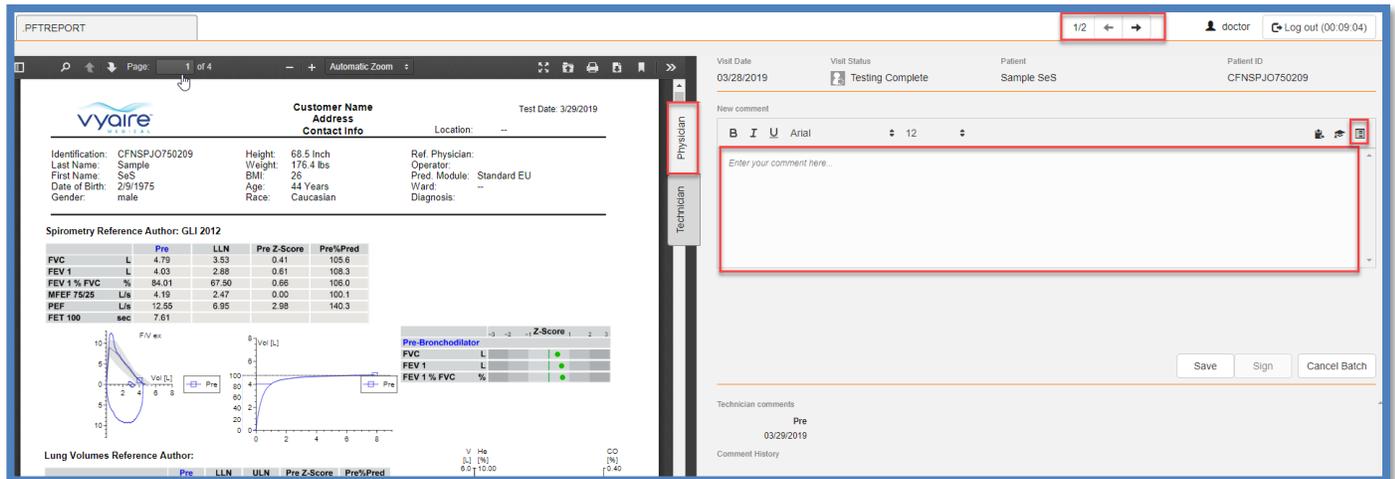

- Ensure the **[Physician]** tab is selected. Free text or dictation software can be used to add an interpretation in the comment area. Clicking on the **[📄]** button opens an area for the selection of predefined interpretative statements. Click **[Sign]** when the interpretation is complete.

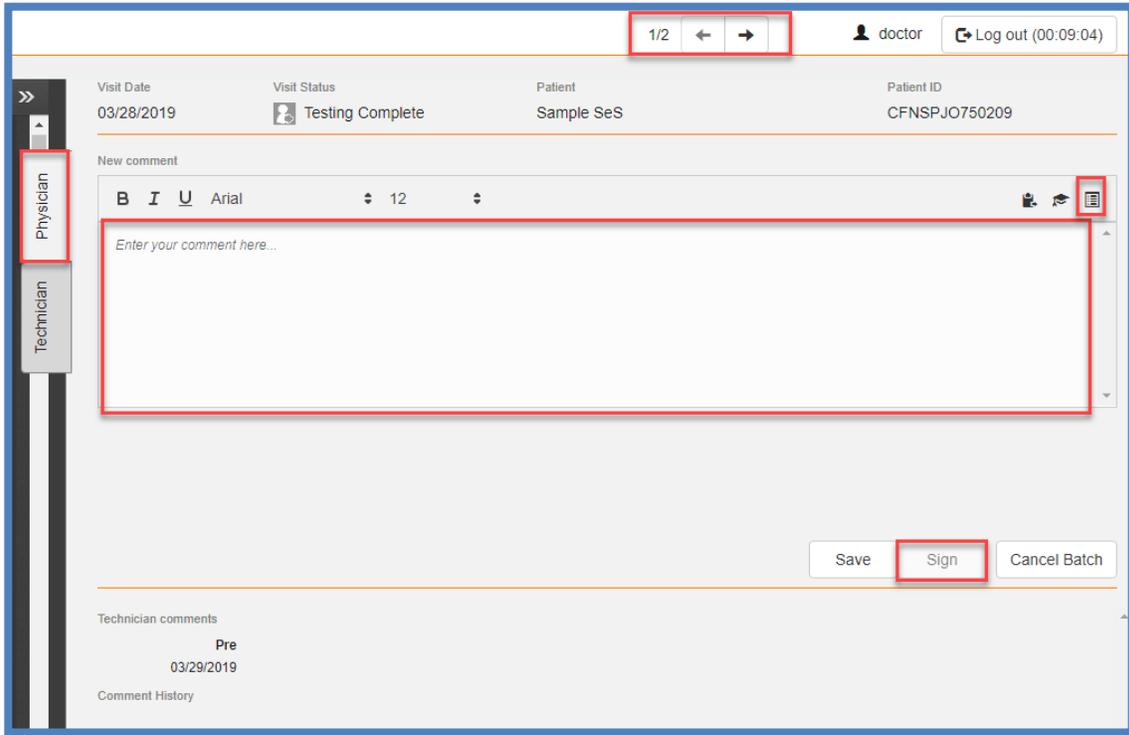


8. Click [**Final**] and [**Sign**] to complete the interpretation.

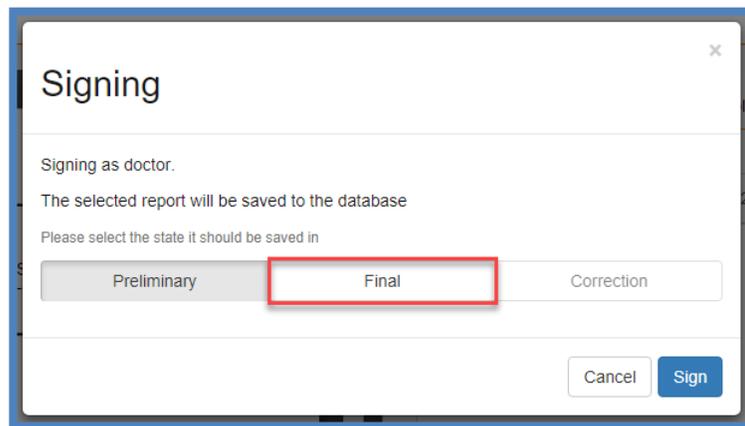

**NOTE:** Once a patient visit is signed, the interpretation becomes locked. Further interpretation comments can be made; however, they will be appended to previously signed interpretations. All signed interpretations will have user and date/time stamp information.

9. If Batch Mode is enabled, the next patient report will automatically populate. If Batch Mode is not enabled the Mobile Review main screen will be displayed.
10. Repeat the steps above until the work list is empty or the batch has been completed.

**NOTE:** If a report interpretation cannot be completed while logged in, click [**Save**] to store your changes.

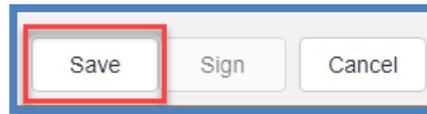

Click [**Update**] to save the changes and update the report.

