

## 1. New ultrasound cleaning agents:

### To Clean Ultrasound:


Please use Orange Top Sani-Cloth Bleach Wipes (preferred cleaning agent and is required for C. diff disinfection).

If bleach wipes are unavailable, Super Sani-Cloth (purple top) and CaviWipes (white top) are permitted.


## 2. Patient search and patient worklist functions:

The MICU ultrasounds now generate a worklist that includes every patient in the MICU, and you can search your patient by name (even first couple letters of name) or MRN.


To search a patient, hit "New Patient"

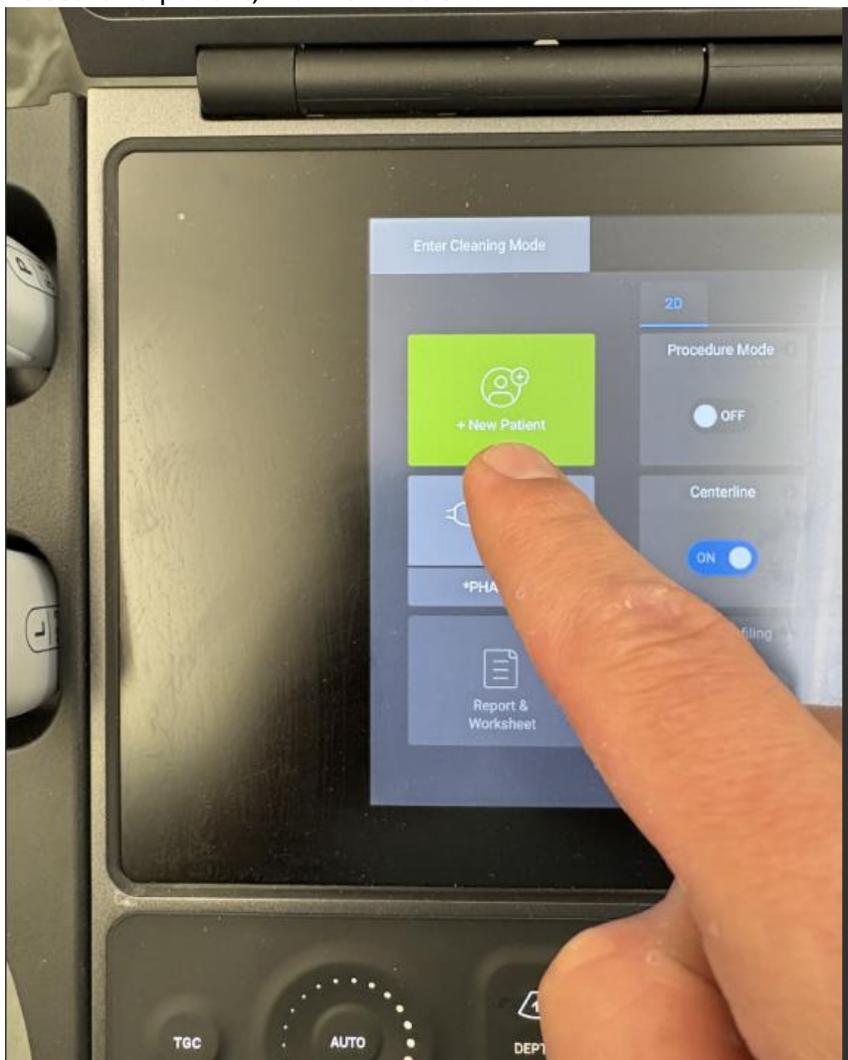


Enter part/all of MRN, first, or last name, and then hit "search"

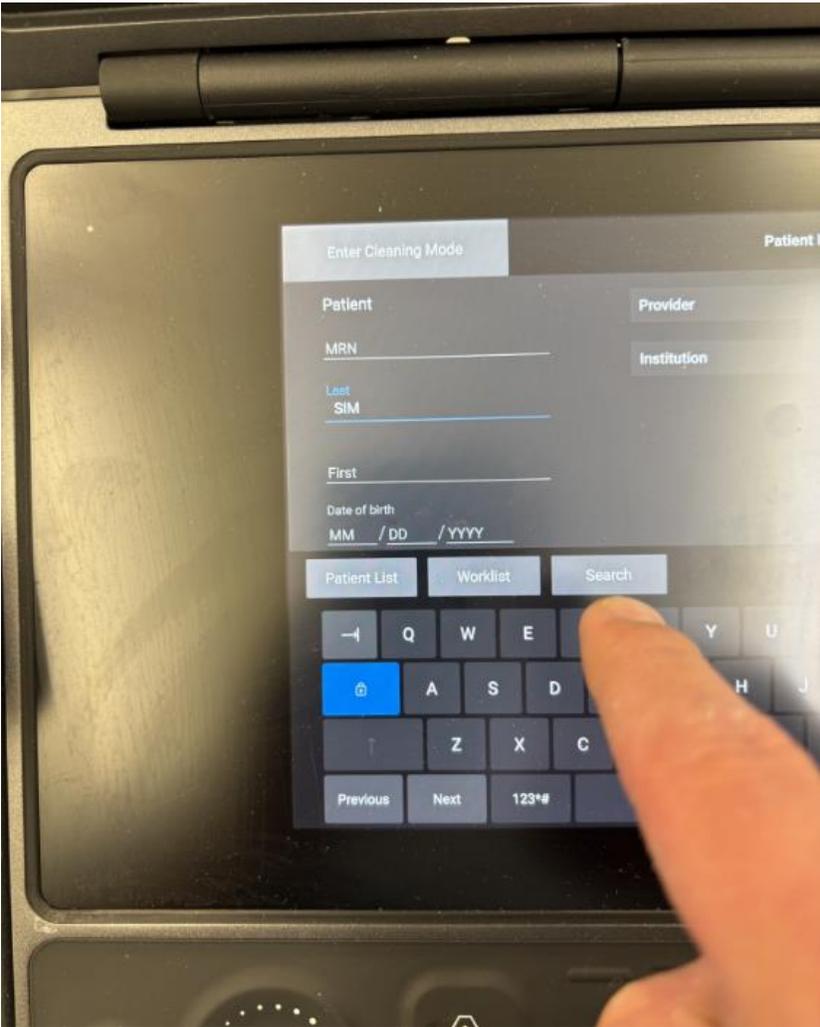


Your patient will then show up on a screen that looks like this (none here since I searched "sim"):

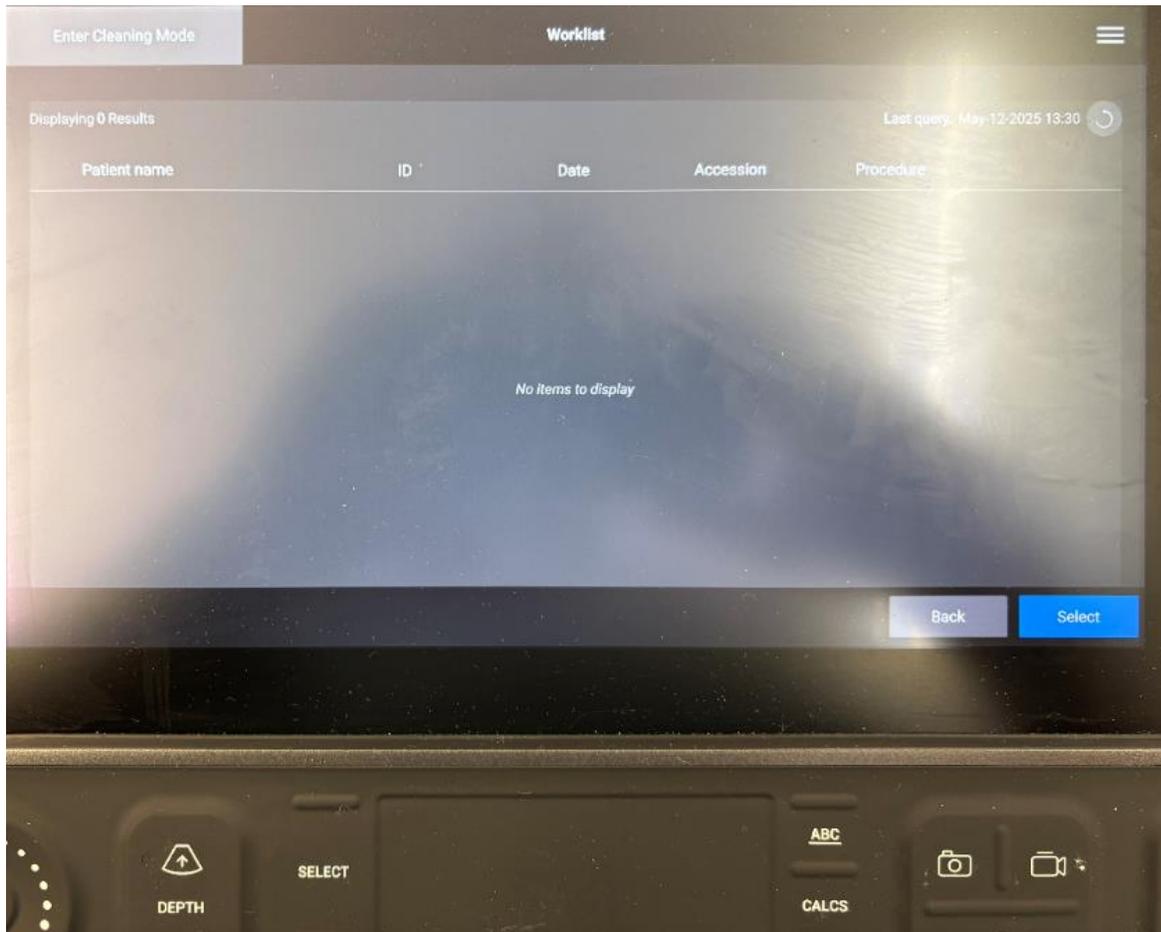

To see the alphabetical-order MICU worklist (which contains every patient in the MICU), select "New Patient," and then select "Worklist" instead of "Search."

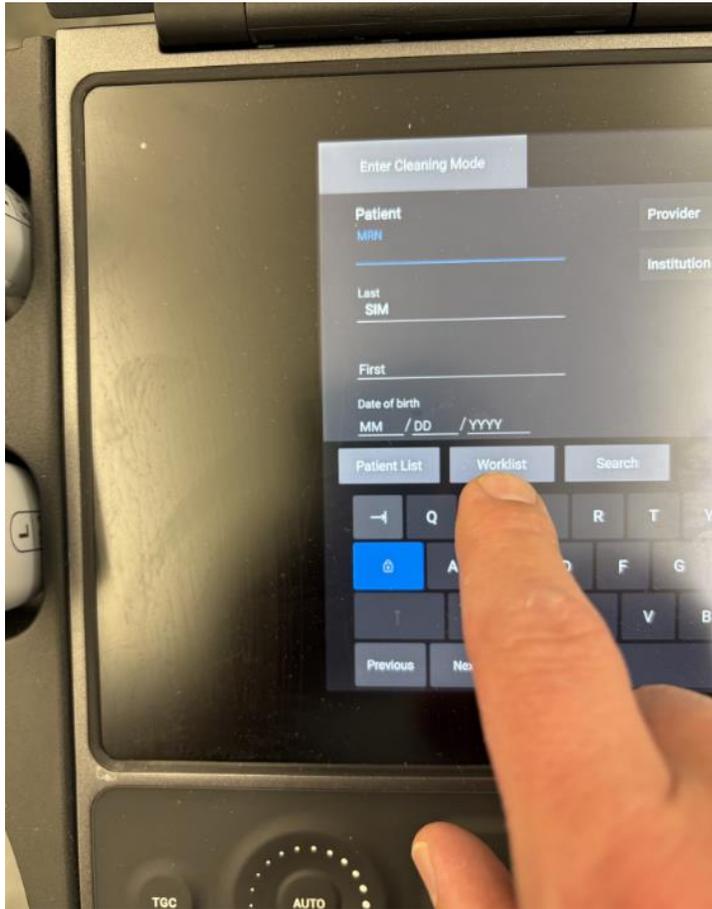

The worklist will populate on a screen that looks like that below (none in picture because of HIPPA/using a sim center unit). **Hit the refresh button** in the top right corner of the screen to get the updated worklist (otherwise defaults to the list that was last queried, which could have been a MICU census from days ago).

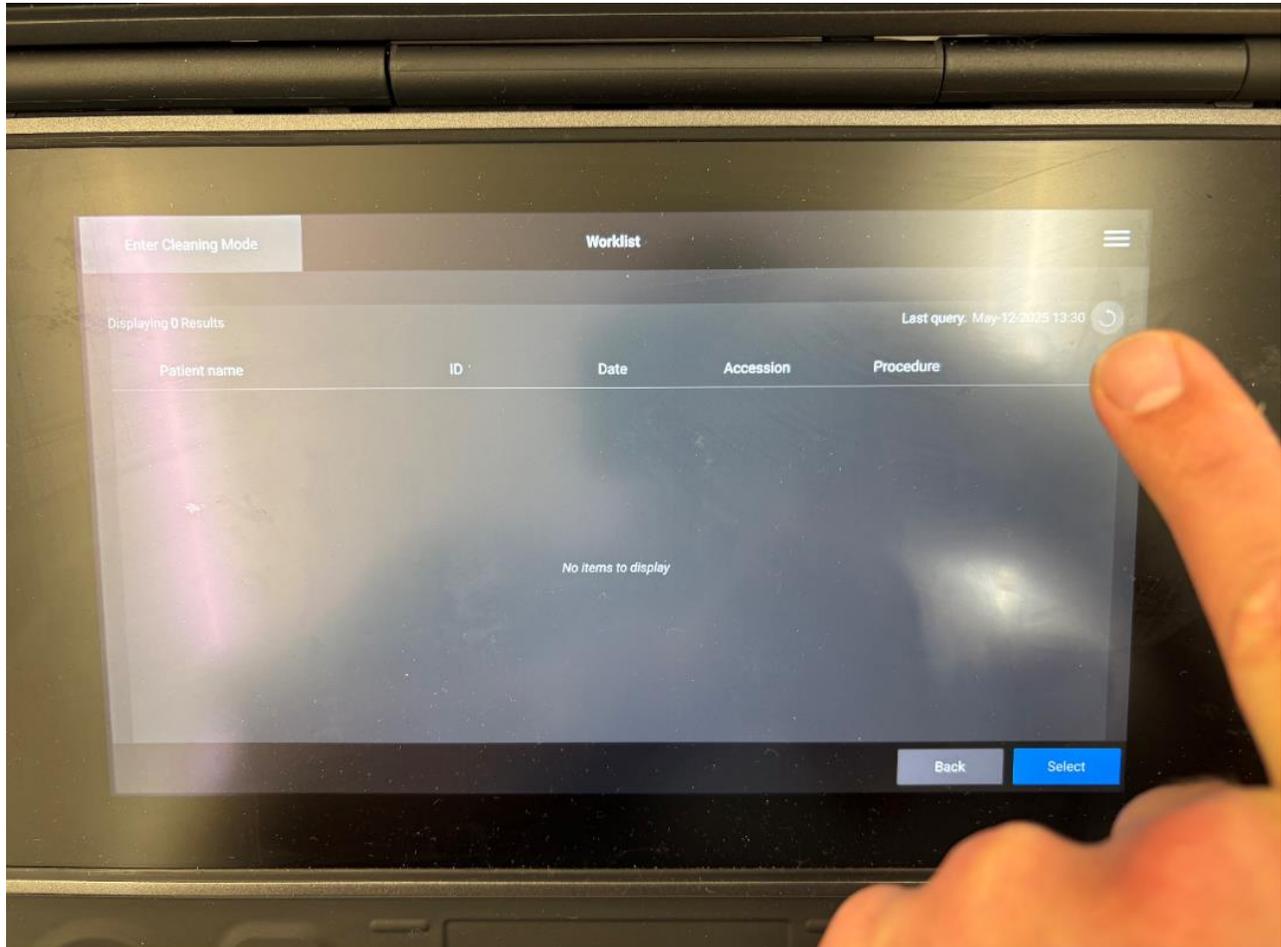

3. If you are in an ICU or in the ED at HFH main hospital, your images should be migrating to QPATHE for medical record, QA, lecture preparation, and/or other applications.

4. Please let me know if there are any issues or questions!